

HUGE



Ear Impression Silicone

PERFIT A/RAPID is a low-viscosity A-Silicone impression material used to make impressions of the external auditory canal. It is widely used in the production of hearing aids, earplugs and other customized ear products.

Incorporating advanced polymerization and self-lubricating technologies, PERFIT A/RAPID ensures rapid setting and easy removal from the ear canal, making it an outstanding material for ear impressions.



Packaging:
· 50 mL × 8 cartridges/box

Advantages



Fast Setting

Extra short time in the ear
(just 90 sec.!))



Low Hardness

Ensures comfortable and
safe experience



High Elasticity

Ensures precise and
accurate impressions



Self-Lubricating

Easy to remove from the
ear canal



High Thixotropy

Easy to handle with
excellent stability




Scannable

Suitable for digital
workflows




Application Procedure:


Designed for precision, comfort, and ease of use — PERFIT A/RA-PID is compatible with cartridge systems, making every step of the impression process efficient and reliable.

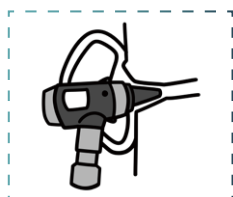


1. Examine the ear

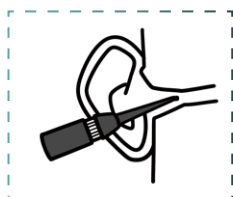


2. Prepare the ear

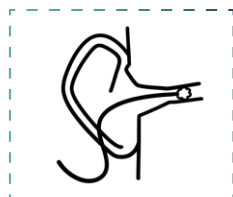

3. Place the impression pad

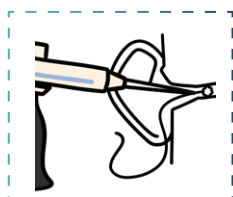


4. Apply the impression material

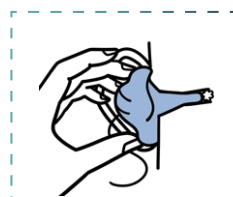

5. Setting time 1:30



6. Finish the impression

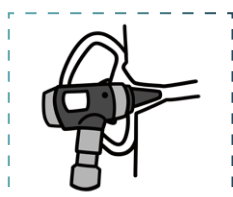

7. Check the ear

90 SEC



Technical Parameters

Technical Parameters	A/RAPID
Working Time (23 °C)	30 s
Setting Time (23 °C)	90 s
Final Hardness (24 h)	40 A
Color	Blue
Shelf Life	3 Years

*Data from HUGE Lab

Order Information Standard Contents:

PERFIT A/RAPID	50 mL × 8 cartridges
Dispensing Gun 1 : 1	1 pc
White Mixing Tips	6 pcs
	50 pcs



Shandong Huge Dental Material Corporation

Add / No.68 Shanhai Road, Donggang District, Rizhao City, Shandong Province, 276800, P.R. China.

Tel / +86 (633) 2277268 marketing@hugedental.com www.hugedental.com



Follow @Huge Dental