

PERFIT

DA DENTAL ADVISOR
Product insights you can trust.

Hydrophilic Impression Materials



With over 10 years of clinical success, PERFIT has become one of the most reliable and widely used impression materials on the market. HUGE continues to expand their range to meet the diverse needs of dental professionals — ensuring there's always a [PERFIT solution for every indication](#).

Outstanding Hydrophilicity & Thixotropy

PERFIT offers excellent flow in moist environments, enabling precise detail reproduction. Its thixotropic properties prevent dripping, making the procedure more comfortable for both dentists and patients.

Excellent Dimensional Stability

Low shrinkage and high dimensional stability allow impressions to remain accurate for up to **28 days** - ideal for both chairside efficiency and lab processing.

Elastic recovery > 99.5%

Linear dimensional change < 0.2%

Smart Handling & Reliable Results

Thanks to its elastic memory, PERFIT can be easily removed from the mouth and returns to its original shape. Available in multiple viscosities and hardness levels, PERFIT provides the flexibility clinicians need to achieve high-precision impressions.

Doctor/Patient-Friendly Working & Setting Times

PERFIT offers efficient curing with sufficient working time and a shorter intraoral setting time. This makes procedures easier for dentists and more comfortable for patients. Faster cross-linking also helps reduce inaccuracies caused by micro-movement.

Optimized Timing for Every Impression Technique

PERFIT Viscosity	Putty		Heavy Body			Mono-phase	Regular Body (Transparent)	Light Body		Super Light Body	A-Silicone for Bite Registration	
	Type 0		Type 1			Type 2	Type 2	Type 3		Type 3	Bite	
Working Time	Normal Set	Fast Set	Dispenser -mix Normal Set	Dispenser -mix Fast Set	Auto-mix Fast Set	Normal Set	Fast Set	Normal Set	Fast Set	Normal Set	Normal Set	Fast Set
	1:30 min.	1:15 min.	2:00 min.	1:30 min.	1:30 min.	2:00 min.	1:30 min.	2:15 min.	1:30 min.	2:15 min.	0:30 min.	0:12 min.
Setting Time in Mouth	3:00 min.	2:30 min.	3:30 min.	3:00 min.	3:00 min.	3:30 min.	3:00 min.	3:30 min.	2:30 min.	3:30 min.	1:30 min.	0:45 min.
Package	· 400 g base · 400 g catalyst · 2 measuring spoons		· 50 mL × 2 cartridges · 50 mL × 2 cartridges + 6 mixers			· 50 mL × 2 cartridges · 50 mL × 2 cartridges + 6 mixers	· 50 mL × 2 cartridges	· 50 mL × 2 cartridges · 50 mL × 2 cartridges + 12 mixers + 12 oral tips		· 50 mL × 2 cartridges · 50 mL × 2 cartridges + 12 mixers + 12 oral tips	· 50 mL × 2 cartridges · 50 mL × 2 cartridges + 6 mixers	

*Data from HUGE Laboratory

HUGE



Shandong Huge Dental Material Corporation

Add / No.68 Shanhai Road, Donggang District, Rizhao City,
Shandong Province, 276800, P.R. China.

Tel / +86 (633) 2277268 marketing@hugedental.com www.hugedental.com



Follow @Huge Dental

v2.0-202507